[Atlantik Genta]

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Combining bone filling and local antibioprofylaxis

ATLANTIK *Genta* is a bi-phased synthetic bone substitute (70% Hydroxyapatite and 30% β -TCP) with antibiotic. It has been specially designed in order to fill in bone cavities for orthopaedic surgical operations (except contaminated surgery), in particular in cases of:

- joint prosthesis,
- spinal,
- traumatology on closed fracture,
- osteotomies,
- filling after tumour ablation.

It is particularly indicated when germs sensitive to gentamicin constitute a possible risk.

ATLANTIK *Genta* Bone substitutes have an optimised porosity and surface to allow a rapid bone ingrowth and a good osteointegration.

The structure of our bone substitute combines at the same time macroporosities whose total interconnection favours the penetration of the cells, and microporosities which increase the surface in contact with the biological fluids.

The ATLANTIK *Genta* range includes granules designed to fill in cavities of complex form, and blocks that can be cut easily by the surgeon in order to adapt to any particular indication.

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Fig.1. Local rate (=synovial liquid) and serum rate (=blood) of gentamicin after implantation of ATLANTIK Genta in the femur of several sheep (average on 5 sheep).

ATLANTIK Genta contains 150 Mg of gentamicin basis, intended to be diffused in the zone operated in the hours which follow the implantation:

- the diffusion lasts less than 48 hours thus avoiding any risk of bacterial resistance.
- the gentamicin dose corresponds to the usual therapeutic dose (3 mg/kg) for a patient of 50 kg.
- effective local concentrations (largely higher than the BMC* of the germs sensitive to gentamicin) can be reached thanks to the dose and the speed of diffusion. Local diffusion has the advantage of generating a short and of limited intensity serum peak (fig.1).

* BMC: Bactericidal minimal concentration

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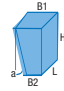
- Packed in double sterile packaging,
- Products delivered with a 18 months minimum sterility limit,
- Store at a temperature < 25°C,
- Use of gentamicin: Precautions for use and counter-indications, refer to the instructions of the manufacturer.

Forme / Shape	Taille / Size*	Quantity	Volume*	Reference
Granulés / Granules 	0,5 mm	1	20 cm ³ ou 10 cm ³	S-G02-G ou S-G01-G
	1 mm	1	20 cm ³ ou 10 cm ³	S-G12-G ou S-G11-G
	2 mm	1	20 cm ³ ou 10 cm ³	S-G22-G ou S-G21-G
	4 mm	1	20 cm ³ ou 10 cm ³	S-G42-G ou S-G41-G
Bloc / Bar 	50 x 30 x 15 mm	1	22,5 cm ³	S-I01-G
	20 x 10 x 10 mm	2	4 cm ³	S-I05-G
	20 x 5 x 5 mm	4	2 cm ³	S-I07-G
Cylindre / Cylinder 	∅ = 30 mm H = 15mm	1	10,6 cm ³	S-I11-G
	∅ = 15 mm H = 15mm	2	5,4 cm ³	S-I122-G
	∅ = 10 mm H = 15mm	2	2,4 cm ³	S-I14-G
Coin / Wedge 	a = 12° B1 = 12 mm B2 = 4,6 mm H = 35 mm L = 30 mm	1	8,7 cm ³	S-I22-G
	a = 10° B1 = 10 mm B2 = 3,8mm H = 35 mm L = 30 mm	1	7,2 cm ³	S-I24-G
	a = 8° B1 = 8 mm B2 = 3,1 mm H = 35 mm L = 30 mm	1	5,8 cm ³	S-I26-G

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Phases	HAP 70% TCP 30%
Impurities	As < 3 ppm Pb < 30 ppm Hg < 5 ppm Cd < 5 ppm

Dimensions	+/- 0,5 mm
Porosity rate	70%
Macroporosity	300 - 600 µm
Compressive Strength	10 Mpa
Gentamicin	150 mg

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